

# Imprint™ 4 Preliminary

VPS Preliminary Impression Material



DARBY DENTAL SUPPLY, LLC



## No dust. No mess. Just automix.

### Indications:

Imprint™ 4 Preliminary is designed for all typical alginate indications such as:

- ▶ Provisional crown and bridge impressions
- ▶ Opposing jaw impressions
- ▶ Study model impressions
- ▶ Preliminary impressions
- ▶ Orthodontic impressions
- ▶ Impressions for bleaching trays and mouth guards

Where alginates require numerous time-consuming, messy work steps, Imprint™ 4 Preliminary VPS material for automatic mixing with the Pentamix™ System makes your procedure faster, cleaner and more convenient — helping to make more accurate preliminary impressions.

### Imprint 4 Preliminary VPS material gives you:

#### ▶ CONTROL OF TIME

**Pouring when convenient** due to long-term dimensional stability

**Multiple pours** for easy remakes of models and provisionals

**Fast-setting “Super Quick”** version available




#### ▶ CONTROL OF PROCEDURE

**High hydrophilicity** for easy model pouring

**Complete disinfection** by submersion


**Scannability** for easy access to the digital workflow

Patient-pleasing **mint flavor**

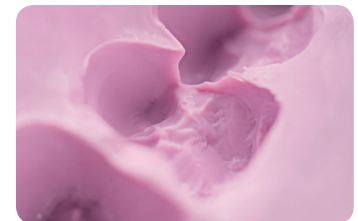

For high-end temporaries and ultimately precise fitting final restorations use Imprint™ 4 Preliminary VPS Material together with Protemp™ Plus Temporization Material.

**3M** ESPE

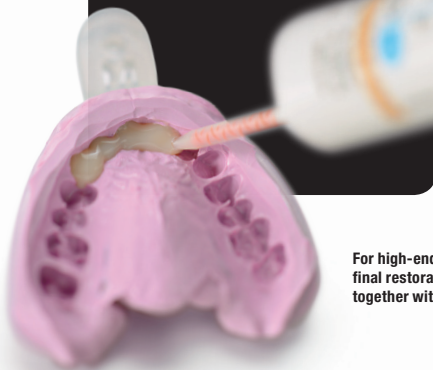

## For preliminary and precision impressions: 3M™ ESPE™ Impression Tray.

With no need for a tray adhesive, the single-use 3M ESPE Impression Tray is ready to use and saves time: Simply select the proper tray size (S, M or L) and fill the tray directly from the mixing device.



		Maximum working time (23 °C)	Constant intra-oral setting time (37 °C)
<b>FAST</b>	<b>Imprint™ 4 Preliminary Penta™ Super Quick</b>	1:00	1:30
<b>REGULAR</b>	<b>Imprint™ 4 Preliminary Penta™</b>	1:30	2:00

## Ordering Information

### Imprint™ 4 Preliminary Penta™ VPS Preliminary Impression Material

Item No.	Product Information
<b>Refills</b>	
8671362 (71521)	<b>Imprint™ 4 Preliminary Penta™</b> 2 base pastes – 300 ml each, 2 catalysts – 60 ml each
8671370 (71522)	<b>Imprint™ 4 Preliminary Penta™ Super Quick</b> 2 base pastes – 300 ml each, 2 catalysts – 60 ml each
<b>Value Packs</b>	
8671360 (71523)	<b>Imprint™ 4 Preliminary Penta™</b> 6 base pastes – 300 ml each, 6 catalysts – 60 ml each
8671368 (71524)	<b>Imprint™ 4 Preliminary Penta™ Super Quick</b> 6 base pastes – 300 ml each, 6 catalysts – 60 ml each

### Imprint™ 4 Preliminary VPS Preliminary Impression Material

Item No.	Product Information
<b>Economy Packs</b>	
8671378 (71539)	<b>Imprint™ 4 Preliminary Regular</b> 10 – 50 ml cartridges
8671380 (71540)	<b>Imprint™ 4 Preliminary Super Quick</b> 10 – 50 ml cartridges
<b>Bulk Packs</b>	
8671382 (71548)	<b>Imprint™ 4 Preliminary Regular</b> 30 – 50 ml cartridges
8671384 (71549)	<b>Imprint™ 4 Preliminary Super Quick</b> 30 – 50 ml cartridges

### Item No. Product Information

<b>3M™ ESPE™ Impression Tray Refills</b> (each containing 10 trays)	
8671184 (71615)	Size S, upper jaw
8671186 (71616)	Size M, upper jaw
8671188 (71617)	Size L, upper jaw
8671190 (71618)	Size S, lower jaw
8671192 (71619)	Size M, lower jaw
8671194 (71620)	Size L, lower jaw
<b>Pentamix™ Lite Automatic Mixing Unit</b>	
* (77907)	Pentamix™ Lite Automatic Mixing Unit incl. 1 Penta™ Cartridge
<b>Pentamix™ 3 Automatic Mixing Unit</b>	
8671251 (77876)	Pentamix™ 3 Automatic Mixing Unit
<b>Accessories</b>	
8671364 (71525)	Imprint™ 4 Preliminary Penta™ Cartridge for Pentamix™ 2
8671372 (71526)	Imprint™ 4 Preliminary Penta™ Super Quick Cartridge for Pentamix™ 2
8671366 (71527)	Imprint™ 4 Preliminary Penta™ Cartridge for Pentamix™ 3
8671374 (71528)	Imprint™ 4 Preliminary Penta™ Super Quick Cartridge for Pentamix™ 3
* (77944)	1 Penta™ Cartridge for Pentamix™ Lite
8671547 (77949)	50 Penta™ Mixing Tips – Red



DARBY DENTAL SUPPLY, LLC

**To Reorder Call 800.645.2310  
or Visit darby.com**

**3M ESPE**

3M, ESPE, Imprint, Penta, Pentamix and Protemp are trademarks of 3M Company or 3M Deutschland GmbH. Used under license in Canada.

©3M 2015. All rights reserved.

70-2013-0661-3 (Rev. C)