



DARBY DENTAL SUPPLY, LLC



**Cool Temp<sup>®</sup>**  
N A T U R A L

### Superior esthetics

- 3 Fluorescent Vita™ shades

### User friendly

- Low shrinkage = better marginal adaptation
- Superior polishability

### Cooler curing

- Low temperature = does not damage or irritate the pulp



**coltène**   
**whaledent<sup>®</sup>**

## PATIENT KEEPS COOL

Cool Temp® NATURAL's low curing temperature does not damage or irritate the pulp to create any discomfort for the patient.

Easy fabrication using Cool Temp® NATURAL also allows less chairtime for the patient.

## DENTIST KEEPS COOL

With Cool Temp® NATURAL, there's no need to worry about making predictable and precise temporaries, without extensive chairside adjustments.

An optimized elastic phase allows the material to be easily removed after a short time in the mouth. The inhibition layer is quickly wiped clean. The result is an accurate provisional that can be trimmed and polished effortlessly.



## EASY AUTOMIX

Cool Temp® NATURAL is supplied in a convenient automix cartridge 50ml that fits the new standard 1:1/2:1 dispenser. We also supply the 5 ml Automix syringe.

Automix delivery is simple, saves time and mess, and consistently provides a homogeneous material.

## STEP-BY-STEP

- Prior to tooth preparation, take an impression. Then prepare the teeth as required.
- Load the impression with Cool Temp® NATURAL and place over the prepared teeth (35 s).
- After 30–45 s in the mouth, remove the Cool Temp® NATURAL provisional. It will have a firm elastic consistency and come out easily.

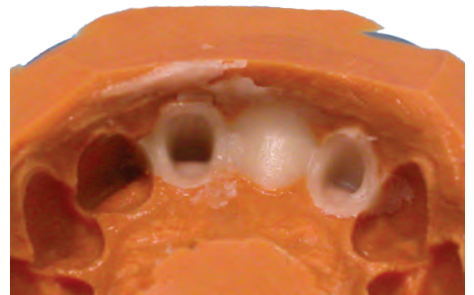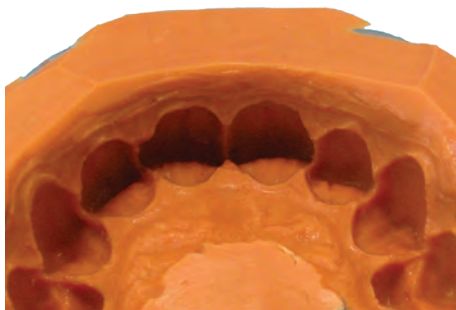

## CLINICAL CASE, DR. AMON GABRIEL SANDER, SCHRUNS, AUSTRIA



The vacuum-formed template prepared on the study model, is filled with Cool Temp® NATURAL.



The vacuum-formed template filled with Cool Temp® NATURAL must be placed into the mouth within 35 s. Excess material should be removed using a plastic instrument.



After 30-45 s in the mouth, Cool Temp® NATURAL can be removed. If necessary, the provisional restoration can be tested to determine if it fits properly.

# bridge material

## 5 ML AUTOMIX SYRINGE

- Effortless delivery
- Precise, direct loading of Matrix
- Less overfill = minimal waste
- No dispenser needed
- Ideal for single crowns



- The complete restoration after preparation & polishing.



### USEFUL TIP:

To further simplify removal, isolate the prepared teeth with Coltène® Separator prior to placing Cool Temp® NATURAL.


### Easy adjustments:

Use Synergy® Flow to fill voids, refine margins and improve contacts.




## STUDY

### Up to 75 % less time in mouth

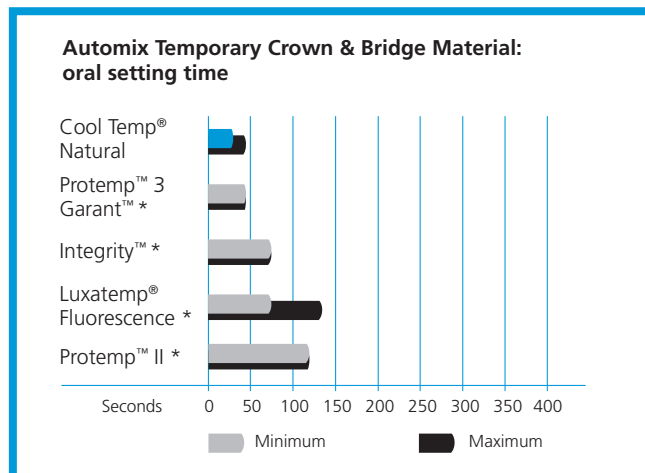

### Faster than leading brands

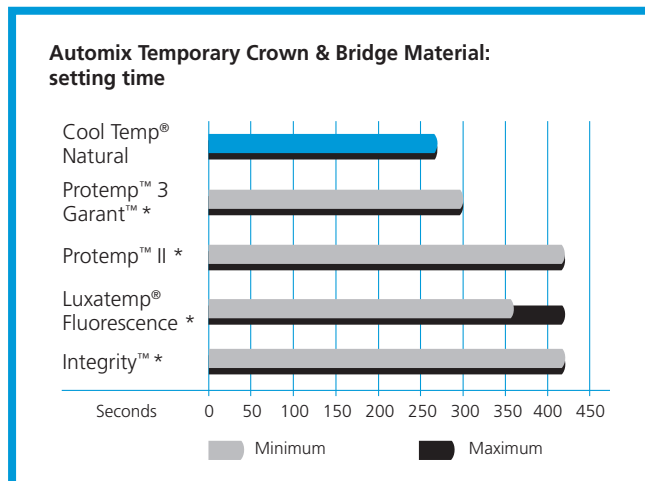

Cool Temp® is a registered brand of Coltène/Whaledent.  
\* All other brands are not registered trademarks of Coltène/Whaledent.



After removing the provisional restoration from the preliminary impression or from the prepared tooth, removal of excess material and preparation of the surface can be performed using rotary instruments. The oxygen inhibition layer on the surface of the Cool Temp® Natural provisional restoration should be removed using alcohol.

Synergy® Flow is ideal to fill voids, refine margins and improve contacts.

Within only a few minutes, a precision provisional is prepared.

# Cool Temp®

NATURAL



## Order Information

### Cool Temp® NATURAL

#### Cartridges

- (C5800) Cool Temp® Intro Kit**  
Cool Temp® cartridge, A2, A3.5 2 x 50 ml  
MT/BROWN/CLEAR/RED/50/P\* 20 pcs.  
SYNERGY® Flow A2/B2 1 x 2.3 g  
SYNERGY® Flow A3.5/B3 1 x 2.3 g  
Flow application needles 4 pcs.
- 9068684**  
**(C5805) Cool Temp® Single Pack**  
Cool Temp® cartridge A2 1 x 50 ml  
MT/BROWN/CLEAR/RED/50/P\* 10 pcs.
- 9068685**  
**(C5810) Cool Temp® Single Pack**  
Cool Temp® cartridge A3.5 1 x 50 ml  
MT/BROWN/CLEAR/RED/50/P\* 10 pcs.
- 9068683**  
**(C5815) Cool Temp® Single Pack**  
Cool Temp® cartridge A1 1 x 50 ml  
MT/BROWN/CLEAR/RED/50/P\* 10 pcs.

#### Syringes

- 9068690**  
**(C5820) Cool Temp® syringe Refill**  
Cool Temp® syringe A1 2 x 5 ml  
MT/BROWN/CLEAR/RED/50/P\* 8 pcs.
- 9068692**  
**(C5824) Cool Temp® syringe Refill**  
Cool Temp® syringe A2 2 x 5 ml  
MT/BROWN/CLEAR/RED/50/P\* 8 pcs.
- (C5825) Cool Temp® syringe Refill**  
Cool Temp® syringe A3.5 2 x 5 ml  
MT/BROWN/CLEAR/RED/50/P\* 8 pcs.

#### Accessories

- 9068688**  
**(C6900) Coltène® Dispenser 50 / 75 ml 1 pc.**
- 9068686**  
**(C7774) MT/BROWN/CLEAR/RED/50/P\* 40 pcs.**
- 9068696**  
**(C6748) MT/BROWN/CLEAR/RED/50/P\* 40 pcs.**
- (C7235) Separator**  
(for intra-oral use) 1 x 5 ml



DARBY DENTAL SUPPLY, LLC

To Reorder Call 800.645.2310  
or Visit [darby.com](http://darby.com)